

# Intercall & Chubb nurse call system connections.

HJR075 alarm system



The HJM301 FLOOR pressure pad can be plugged directly into the nurse call system (does not require the HJR075 alarm system) by using the HJM308 cable

HLV301M Floor pad sensor



HJM308 cable

HJM330.M Nurse call handset

HLV006 Y connector

HJM309 cable



HLV006 Y connector



HLV301M

Mattress pad sensor



HLV301S

Seat pad sensor.



1. Connect a HLV006 Y connector into nurse call system
2. Connect nurse call handset to one connector and a HJM309 cable to the other connector
3. Connect the other end of the HJM309 cable into the HJR075 alarm system socket marked NURSE CALL
4. Connect another HLV006 Y connector to the black cable already connected to the Alarm system.
5. Connect the HLV301M mattress pad to one connector and the HLV301S seat pad to the other.
6. If the chair is located away from the bed you can use a HLV006E 6-meter extension cable

- Carry out a risk assessment to ensure this system is suitable for intended use and test system for correct operation each time before use.
- If using a seat pad and mattress pad only one can be in use at any time.
- Read the HJR075 alarm system instructions carefully.
- If you have any doubts about the above please contact Harvest Healthcare LTD

**Stereo jack-plug**

A stereo jack plug can be identified by the 2 black insulation bands



**Mono jack-plug**

A mono jack plug can be identified by the single black insulation band

**HJM308 CABLE**

6.35MM mono jack-plug with 3.5mm mono jack-plug.

Used to connect the floor mat to the nurse call system



**HJM309 CABLE**

6.35MM mono jack-plug with 6.35mm mono jack-plug.

Used to connect the HJRO75 alarm system to the nurse call system



**HJM330.S**

6.35MM mono jack-plug with to nurse call button



**HLV006 Y connector**

6.35mm stereo jack-plug to 2 x 6.35mm sockets.

Used to connect either mat and nurse call button to nurse call system

Or seat and mattress sensors to nurse call system

